Heartworm Antigen Test Kit

For veterinary use only

English version

SNAP® HTWM

In-vitro diagnostic for the semiquantitative detection of *Dirofilaria immitis* (*D. immitis*) antigen in canine and feline whole blood, serum or plasma.

- Precautions and warnings**
- All waste should be properly decontaminated prior to disposal.
 - Do not mix components from kits with different serial numbers.
 - Do not use a SNAP device that has been activated prior to the addition of sample.
 - Refer to country specific Material Safety Data Sheet for regional hazard identification.

- Storage**
- Store at 2–25°C until expiration date.
 - All components must be at room temperature (18–25°C) before running the test.

Kit components		
Item	Reagents	Quantity
1	1 bottle anti-HTWM: HRPO conjugate (Preserved with ProCin™ 150)	7.0 mL
2	SNAP Device	5, 15 or 30
Reagents contained in each device:		
Wash solution (preserved with ProCin™ 150)		0.4 mL
Substrate solution		0.6 mL
Other Components: transfer pipettes, sample tubes, and reagent rack		

Conjugate - H317 / H402 / H412 / P261 / P280 / P333+P313: May cause an allergic skin reaction. Harmful to aquatic life. Harmful to aquatic life with long lasting effects. Avoid breathing mist/vapors. Wear protective gloves. If skin irritation or rash occurs: Get medical advice/attention.

Wash - H317 / H319 / H402 / H412 / P261 / P280 / P305+P351+P338 / P337+P313 / P333+P313: May cause an allergic skin reaction. Causes serious eye irritation. Harmful to aquatic life. Harmful to aquatic life with long lasting effects. Avoid breathing mist/vapors. Wear eye protection/face protection. Wear protective gloves. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention.

WARNING

- Sample information**
- Samples must be at room temperature (18–25°C) before beginning the test procedure.
 - Serum, plasma or anticoagulated whole blood (e.g., EDTA, heparin), either fresh or stored at 2–8°C for up to one week, can be used.
 - For longer storage, serum or plasma can be frozen (-20°C or colder) and then re centrifuged before use.
 - Hemolyzed or lipemic samples will not affect test results.

- Test procedure**
- If stored in a refrigerator, allow all components to equilibrate at room temperature (18–25°C) for 30 minutes. **Do not heat.**

2


3

4


5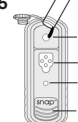

The sample will flow across the result window, reaching the activation circle in 30–60 seconds. Some sample may remain in the sample well.


6

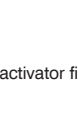Note: Some samples may not flow to the activation circle within 60 seconds, and, therefore, the circle may not turn color. In this case, press the activator after the sample has flowed across the result window.

7

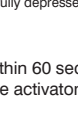- Interpreting test results**
- Positive result**
- Any amount of color development in the sample spots indicates a positive result and color intensity is proportional to the heartworm antigen concentration in the sample.

Low antigen levels

High antigen levels

Positive Control

High antigen indicator

Low antigen indicator

Negative control

Negative result

Only the positive control spot develops color.

- Invalid results**
- Negative Control (safeguard against false positives)** — If color in the negative control spot is equal to or darker than the color in the sample spots, the result is invalid and the sample should be retested.
 - No Color Development** — If the positive control does not develop color, repeat the test.
 - Background** — If the sample is allowed to flow past the activation circle, background color may result. Some background color is normal. However, if colored background obscures the test result, repeat the test.


IDEXX SNAPshot Dx® Analyzer

Canine test results can also be read using the SNAPshot Dx Analyzer. A complete description of how to enter patient data and read test results using the SNAPshot Dx Analyzer can be found in the SNAPshot Dx Analyzer Operator's Guide.

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Symbol descriptions	
	Use by date
	Batch code (lot)
	Serial number
	Temperature limitation
	Manufacturer
	Catalog number
	Authorized representative in the European Community
	Consult instructions for use
	In vitro diagnostic
	Date of manufacture

IDEXX Technical Support
USA/Canada: 1 800 248 2483 • idexx.com
Australia: 1300 44 33 99 • idexx.com.au
Europe: idexx.eu

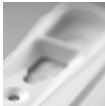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